

talupro® Ankle Brace

☎ 00800 03032900

Ankle orthosis for stabilisation in one plane,
adjustable through pronation strap

Indications

- Postsurgical protection after ligament repair/reconstruction
- Conservative therapy of ankle joint distortions of all degrees of severity
- Functional therapy of ligament rupture
- Ligament injuries of the ankle joint
- Severe ankle joint distortion with loosened joint
- Conservative or functional follow-up treatment of ankle joint fractures and ligament ruptures

Features

- Anatomically shaped shells
- Air chambers made out of skin-friendly polyurethane sheeting
- Air chambers' inner core made out of PU foam filling
- PVC-free and free of plasticizers
- Available in white and black
- Pronation strap is easily removable

Mode of Operation

The talupro® provides medial and lateral support to the ankle joint while allowing a great degree of flexion and extension. This allows for a normal motion sequence. The air chambers cushion the supports to prevent unpleasant pressure. Stronger padding at both ankles provides higher pressure at the bottom third of the supporting area. The applied pressure decreases from the bottom towards the top, helping to reduce swelling more rapidly.

The pronation strap provides additional stability to the forefoot. The pronation strap can be removed. The pronation strap consists of different firm and semi-elastic materials. The inelastic materials have a supporting and fixating effect. The pronation strap is wrapped around the foot according to the principles of taping in order to counteract supination and talus prolapse/forward movement of the talus.



Functional
orthoses

Size chart and article numbers

Size	Body height	Colour	Art.-No. left	Art.-No. right
S	under 158 cm	white	610S-L	610S-R
M	158 - 163 cm	white	610M-L	610M-R
L	from 163 cm	white	610L-L	610L-R
M	to 163 cm	black	610MS-L	610MS-R
L	from 163 cm	black	610LS-L	610LS-R

Measurement Chart

Noch nicht richtig übersetzt

